

CORRECT MEASUREMENT OF PHILADELPHIA COLLARS

Measure the circumference of neck(inches)

This measurement will give you the *size* of collar tht you require.

- **Small** = 10 - 13 inches
- **Medium** = 13 - 16 inches
- **Large** = 16 - 19 inches

Measuring for the correct height and fit

1. Measure from the **prominence of the chin to the sternal notch**(inches)
2. Choose the sizing that is closest to that measurement

Available sizes

- 2¹/₄(F1 only)
- 3¹/₄
- 4¹/₄
- 5¹/₄

Created: 8/105 by Dr K Havill Reviewed: 8/105 by Dr K Havill Guideline to be reviewed before : 6/107

Disclaimer: These guidelines are intended for and to be used only by experienced critical care staff under direct supervision of Hunter Health Area Intensive Care Specialists in designated Hunter Health Area Critical Care Areas. The Authors will not be responsible for inappropriate use of these guidelines.